Remar

— Dental —

Trust Remar For Total Dental Care

FELIX

LUCAS

KLAUS

ANTON

MARCUS

— 2025

www.remardental.de

Remar

Dental

Trust Remar For Total Dental Care

www.remardental.de

[remardental](https://www.instagram.com/remardental)

[remardental](https://www.facebook.com/remardental)

MADE IN GERMANY

REMAR

ANTON Selective Etch Bond

Self-etching, light-curing one-component bonding in only one step.

DESCRIPTION

Anton Selective Etch Bond is a single-component, self-etching, light-cured dental adhesive which enables filling composites to be safely and reliably bonded onto natural dental enamel. The adhesive can be used in both self-etch and selective-etch techniques.

Anton Selective Etch Bond is compatible with all conventional light-cured composite film materials.

COMPOSITION

2-(hydroxyethyl) methacrylate, urethane dimethacrylate, 4-(2-methacryloyloxyethyl) trimellitic anhydride, diphenyl(2,4,6-trimethylbenzoyl) phosphine oxide

INDICATIONS

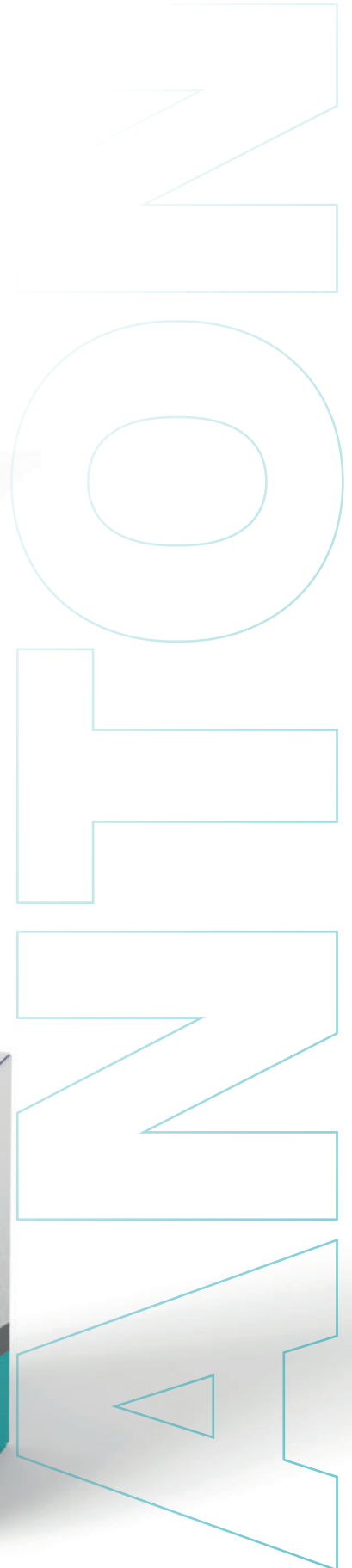
- Direct, light-cured composite restorations onto natural teeth.
- Indirect restorations made from composite and ceramic (inlays, onlays, veneers, crowns) with light-cured and dual-cured luting composites e.g. with Klaus Root Cem Viscous.
- Repairs to fractured composite fillings.

PACKAGING

+RMR /M/100544 5ml-Bottle

CONTRAINDICATIONS

- Do not bring into contact with exposed pulp tissue.
- Do not use if there is insufficient lighting (e. g. when fastening root canal posts).

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2-Hydroxyethylmethacrylat, Urethane dimethacrylate, 4-(2-Methacryloyloxyethyl)-trimellitsäureanhydrid, Diphenyl(2,4,6-trimethylbenzoyl) phosphinoxid

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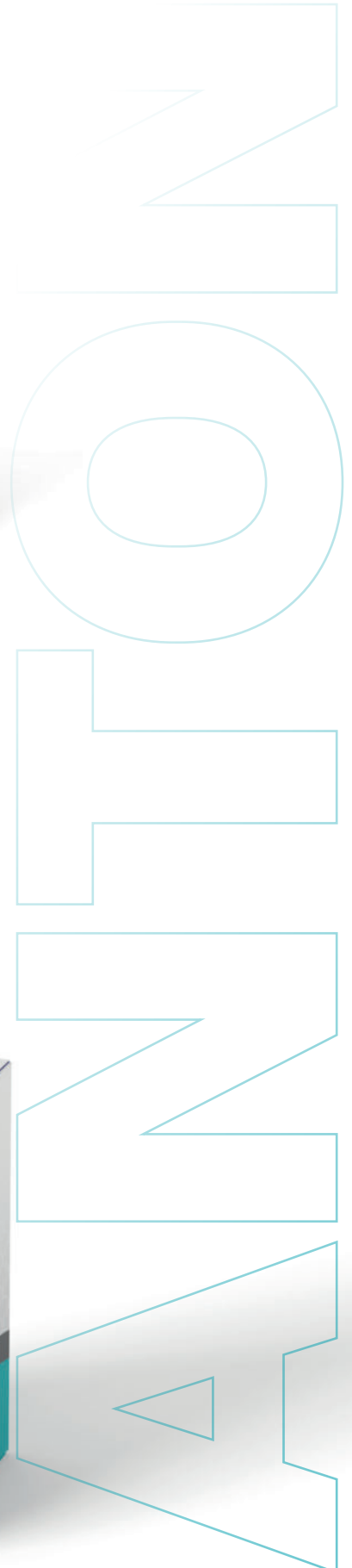
Light-curing, radiopaque posterior composite for bulk-fill technique

DESCRIPTION

Felix Bulk Fill Composite is a light curing, flowable, highly radiopaque (210 % Al) composite for posterior restorations, for direct filling and the „bulk fill“ technique. Felix Bulk Fill Composite is available in syringes and compules. The compules are for single use. Please do not reuse them, as this makes it impossible to rule out contamination and germ formation. The product complies with the guideline DIN EN ISO 4049.

COMPOSITION

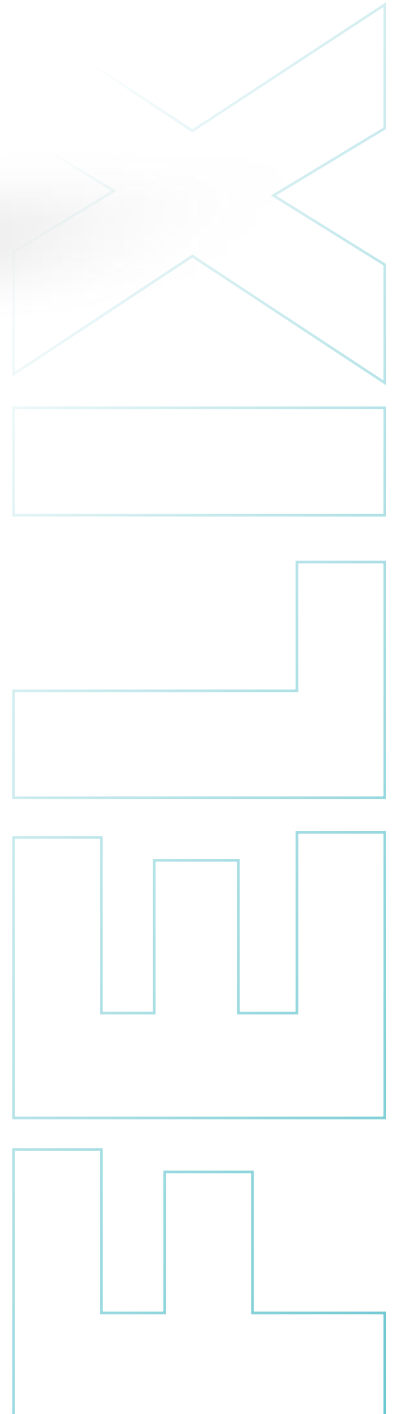
Glass powder, , silicon dioxide, 1,4-butanediol dimethacrylate
 Filler content: 77% by weight (57% by volume) inorganic fillers (0.005 - 40 µm)

PACKAGING

+RMR /M/100542	2x2g-syringe + 10 cannulas
+RMR /M/100543	2x2g syringe + 10 cannulas Medium

INDICATIONS

- Fillings with layer thicknesses up to 4 mm in Black's class I cavities
- Base fillings/liners with layer thicknesses up to 4 mm in Black's class II cavities. For the following layers in a class II cavity, only use composites which are suited to filling Black's class II cavities.
- Cavity lining – as a first (bottom) layer in Black's class I and II cavities

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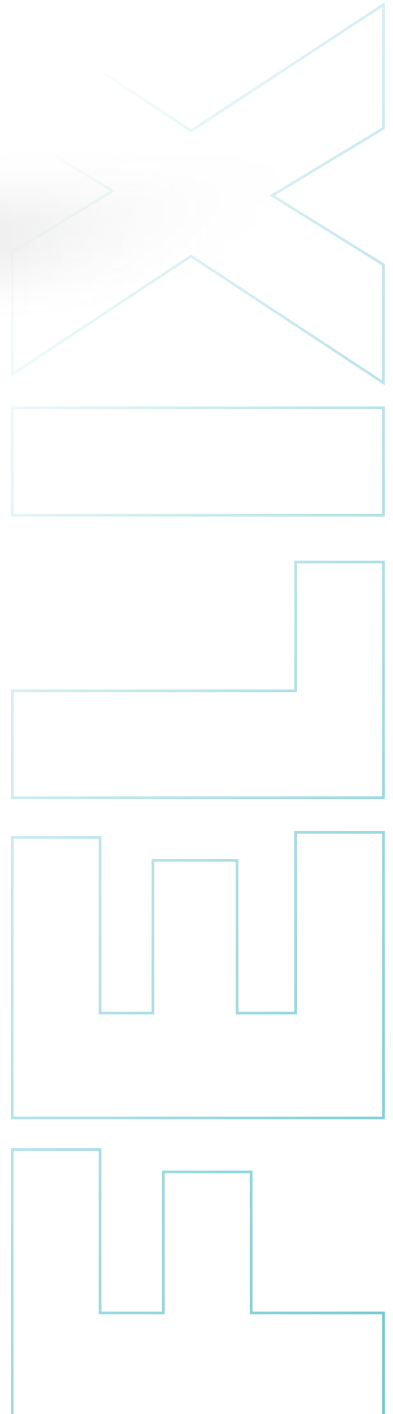
DESCRIPTION

FELIX Flow Viscous Composite is a light-curing, flowable, highly radiopaque (210% Al) composite with a high viscosity. The guidelines and requirements of DIN EN ISO 4049 apply. FELIX Flow Viscous Composite is available in syringes and compules. The compules are for single use only. Please do not reuse them, as this makes it impossible to rule out contamination and germ formation.

COMPOSITION

Glass powder, diurethane dimethacrylate, tetramethylene dimethacrylate, silicon dioxide
Total filler: 77% by weight (57% by volume) anorganic filler (0.005–40µm)

PACKAGING

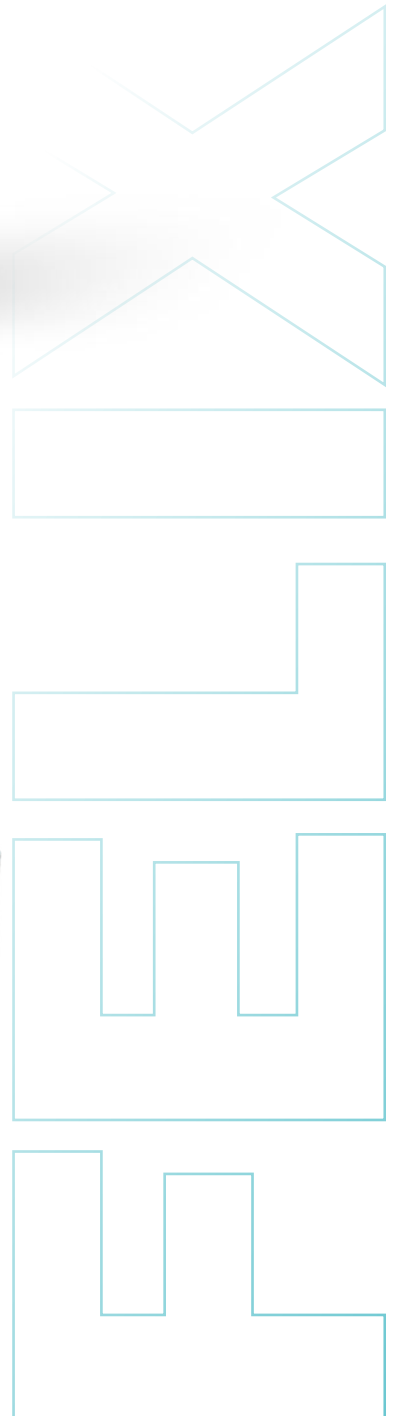
+RMR /M/100534	2X2g-syringe +10 cannulas A1
+RMR /M/100535	2X2g-syringe +10 cannulas A2
+RMR /M/100536	2X2g-syringe +10 cannulas A3
+RMR /M/100***	2X2g-syringe +10 cannulas A3.5
+RMR /M/100***	2X2g-syringe +10 cannulas B1
+RMR /M/100***	2X2g-syringe +10 cannulas B2
+RMR /M/100***	2X2g-syringe +10 cannulas B3
+RMR /M/100***	2X2g-syringe +10 cannulas CLEAR

INFORMATION ON PROCESSING

Light intensity for the curing	≥ 650 mW/cm ²
Wavelength for the curing	350 - 500 nm
Curing time	40 sec.

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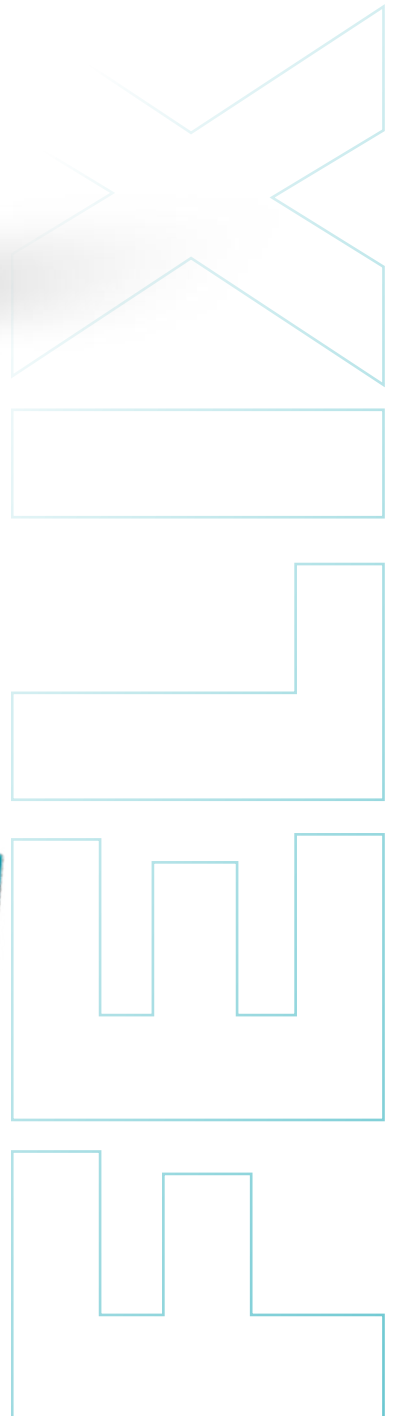
DESCRIPTION

Felix Flow Viscous Composite is a light-curing, flowable, highly radiopaque (210% Al) composite with a high viscosity. The guidelines and requirements of DIN EN ISO 4049 apply. Felix Flow Viscous Composite is available in syringes and compules. The compules are for single use only. Please do not reuse them, as this makes it impossible to rule out contamination and germ formation.

COMPOSITION

Glass powder, diurethane dimethacrylate, tetramethylene dimethacrylate, silicon dioxide
 Total filler: 77% by weight (57% by volume) anorganic filler (0.005–40µm)

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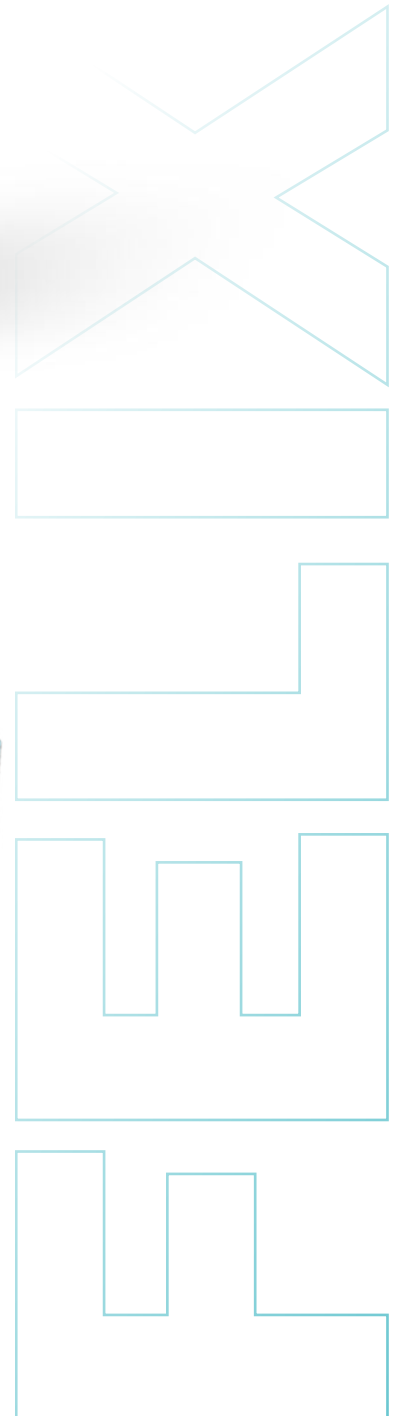
FELIX Flowable Composite

The basic flowable for minimally invasive restorations

DESCRIPTION

Felix Flowable Composite is a light-curing, flowable, radio-paque composite with low viscosity.

COMPOSITION

Glass powder, diurethane dimethacrylate, Bis-GMA, tetramethylene dimethacrylate, splitter polymerisate Filler content: 55% by weight (36% by volume) inorganic fillers (0.005 - 3 µm)

PACKAGING

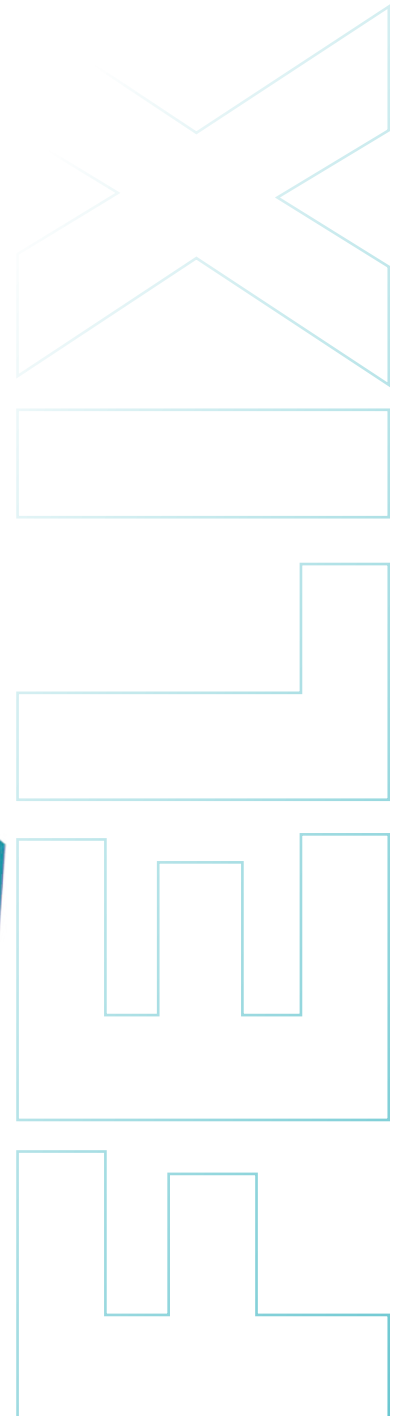
+RMR /M/100***	XXXXX A1
+RMR /M/100***	XXXXX A2
+RMR /M/100***	XXXXX A3
+RMR /M/100***	XXXXX A4
+RMR /M/100***	XXXXX B2
+RMR /M/100***	XXXXX B3
+RMR /M/100***	XXXXX B4
+RMR /M/100***	XXXXX C3

INFORMATION ON PROCESSING

Light intensity for the curing	≥ 650 mW/cm ²
Wavelength for the curing	350 - 500 nm
Curing time	40 sec.
Maximum layer thickness	2 mm

INDICATIONS

- Fissure sealing
- Extended fissure sealing on molars and premolars
- Fillings of class V according to Black (cervical caries, root erosions, wedge-shaped defects)
- Minimally invasive fillings of classes I and II according to Black in the area not subject to significant chewing stress
- Minimally invasive fillings of class III according to Black
- Correction of dental enamel defects
- Blocking out of undercuts
- Minor shape and colour corrections on the enamel

FELIX Flowable Composite

New generation of a light-curing and radiopaque flow composite

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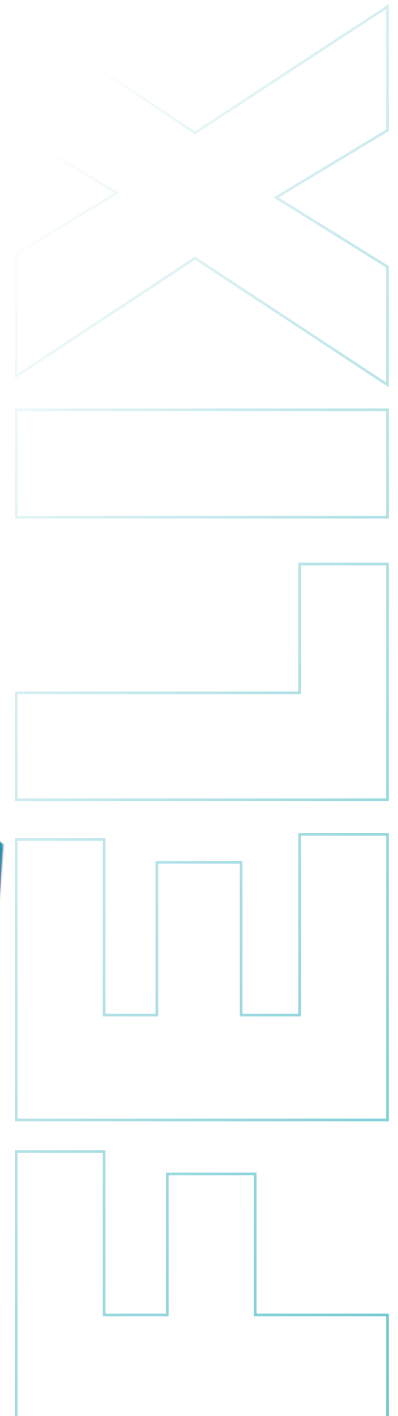
FELIX Micro Hybrid Composite

Universal light-curing filling composite perfect for restorations in both the anterior and posterior tooth areas.

DESCRIPTION

FELIX Micro Hybrid Composite is a light curing, hybrid composite containing an ultrafine, radiopaque glass filler and is indicated for placing fillings using adhesive techniques. It can be polished to a high lustre. Due to the ultrafine particle filler, extremely homogeneous restorations can be placed which are easily polished to a high lustre. The chameleon effect matches the shade of the filling perfectly to the tooth structure. The guidelines of EN ISO 4049 have been complied with. FELIX Micro Hybrid Composite is available in syringes and compules. The compules are for single use. Please do not reuse them, as this makes it impossible to rule out contamination and germ formation.

COMPOSITION

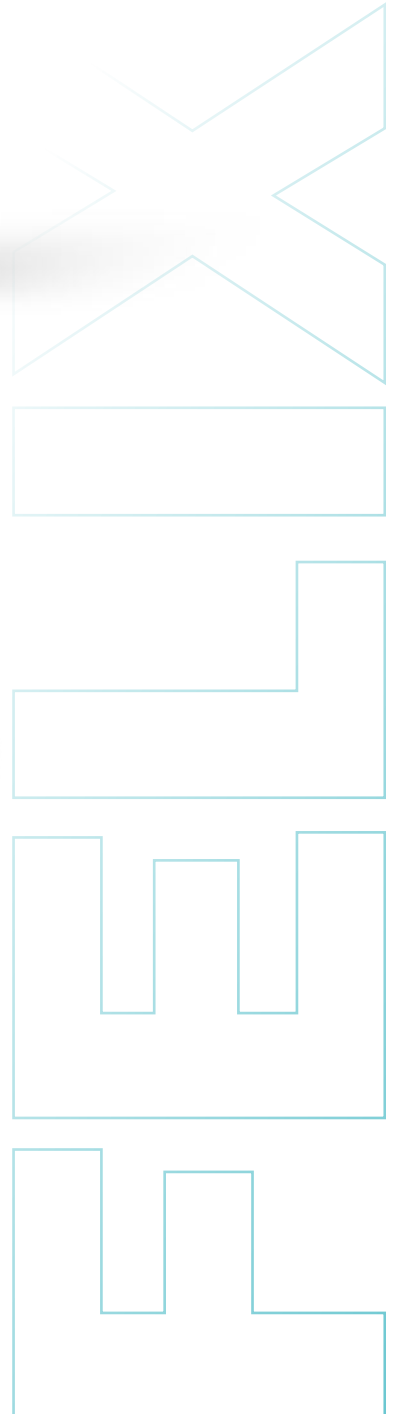
Glass powder, diurethane dimethacrylate, silicon dioxide, Bis-GMA, tetra-methylene dimethacrylate.
Total filler: 75% by weight (53% by volume) anorganic filler (0.005 - 3.0 µm)

PACKAGING

+RMR /M/100527	4g syringe A1
+RMR /M/100528	4g syringe A2
+RMR /M/100529	4g syringe A3
+RMR /M/100530	4g syringe UNIVERSAL

INDICATIONS

- Direct anterior and posterior restorations in Black's classes I, II, III, IV, and V cavities.
- Inlays, onlays and laminate veneers
- Extended fissure sealing in molars and premolars
- Stump build-up
- Splinting loose teeth
- Adjusting the contours and shades to improve aesthetics

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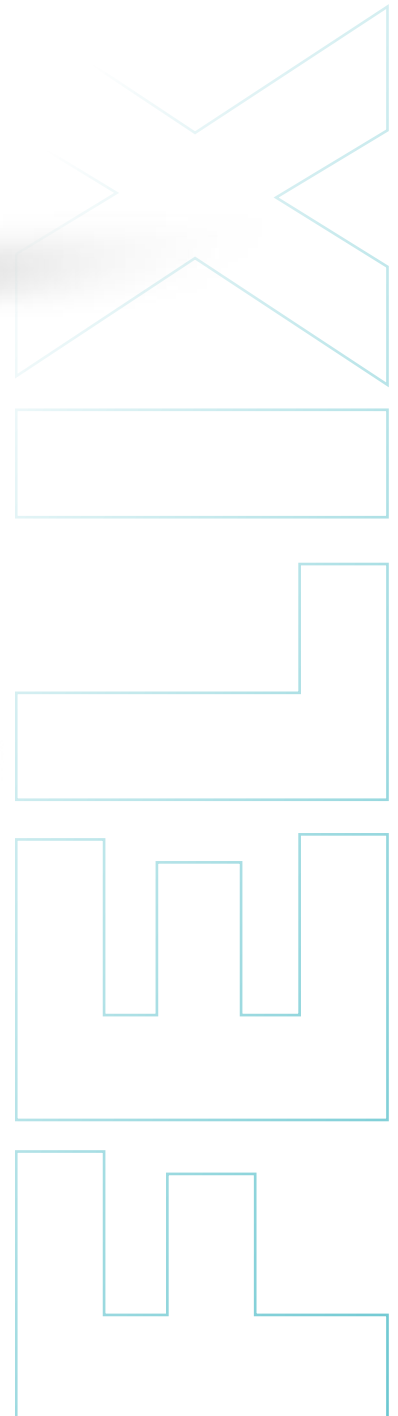
INDIKATION

- Fissure sealing
- Extensive fissure sealing on molars and premolars
- Fillings in Black's class V (cervical caries, root erosions, v-shaped lesions)
- Minimally invasive fillings in Black's classes I, II and III
- Corrections of enamel defects
- Blocking out undercuts
- Small corrections of shape and color of the enamel

FELIX Nano Composite

The multi-talent for long-lasting fillings
Light-curing Nanocomposite

DESCRIPTION

FELIX Nano Composite is a light-curing nano-composite for the adhesive filling technique. It contains an ultrafine, radiopaque glass filler. The composition with ultrafine filler leads to exceptionally homogenous restorations. An optimal adaption of the color is the result of an optimally adjusted chameleon effect. The guidelines and requirements of DIN EN ISO 4049 apply. FELIX Nano Composite is available in syringes and compules. The compules are for single use only. Please do not reuse them, as this makes it impossible to rule out contamination and germ formation.

COMPOSITION

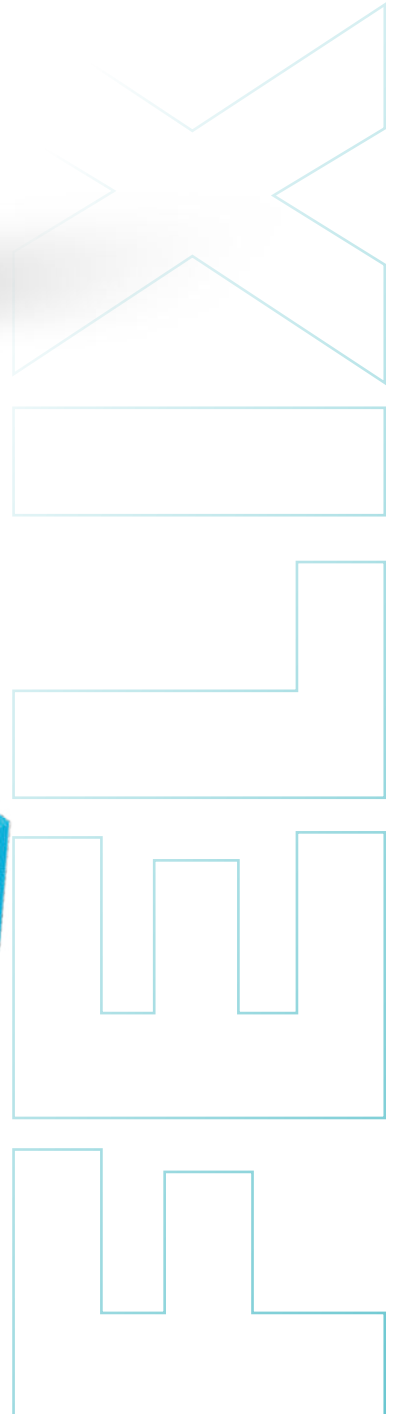
Glass powder, silicon dioxide, diurethane dimethacrylate, BisGMA, splitter polymerizate, tetramethylene dimethacrylate
Filler content: 83.5% by weight (66.5% by volume) inorganic fillers (28 nm - 40 µm)

PACKAGING

+RMR	/M/100524	4g syringe A1
+RMR	/M/100525	4g syringe A2
+RMR	/M/100526	4g syringe A3
+RMR	/M/100561	4g syringe B1
+RMR	/M/100562	4g syringe B2
+RMR	/M/100563	4g syringe C2
+RMR	/M/100564	4g syringe Opaque A2
+RMR	/M/100565	4g syringe Opaque B2
+RMR	/M/100539	Compules 20x0.3g A1
+RMR	/M/100540	Compules 20x0.3g A2
+RMR	/M/100541	Compules 20x0.2g A3

INDICATIONS

- Direct anterior and posterior tooth restorations in Black's classes I, II, III, IV and V
- Inlays, onlays and veneers
- Extended fissure sealing on molars and premolars
- Core build-up
- Splinting of loosened teeth
- Corrections of shape and color to enhance aesthetics

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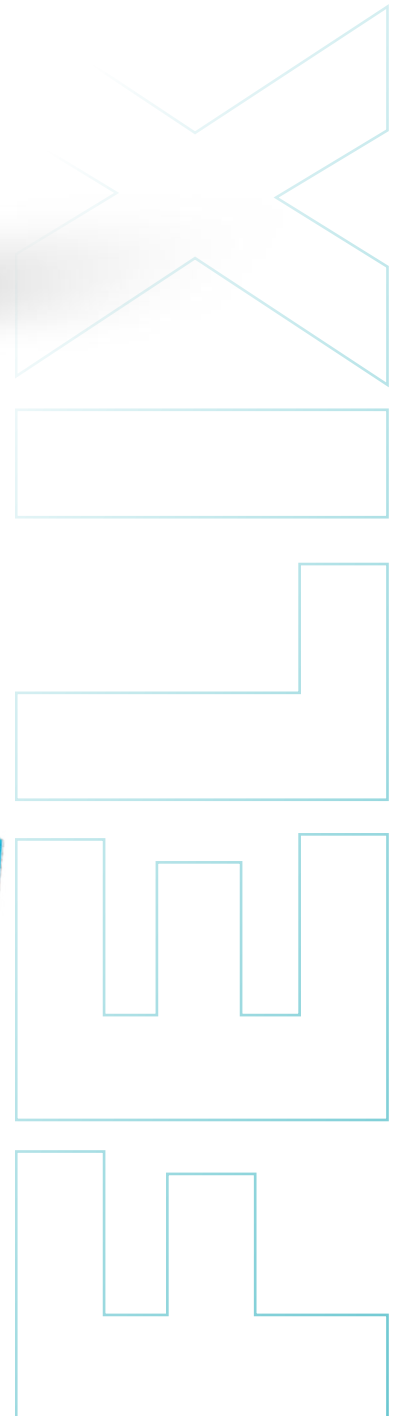
FELIX Bio Composite

Universal light-curing filling composite perfect for restorations in both the anterior and posterior tooth areas.

DESCRIPTION

Felix Bio Composite is a light curing radio-opaque polymer-based dental filling composite for making restorations using adhesive filling techniques. It follows the standard ISO 4049. Bio Composite provides good handling and sculptability characteristics. Ideal for use in anterior and posterior restorations.

COMPOSITION

- Glass powder, diurethane dimethacrylate, silicon dioxide, tricyclodecane dimethanol dimethacrylate
- Filler content: 81 weight-% (77 vol.-%) inorganic fillers (0,005 - 40 µm)

PACKAGING

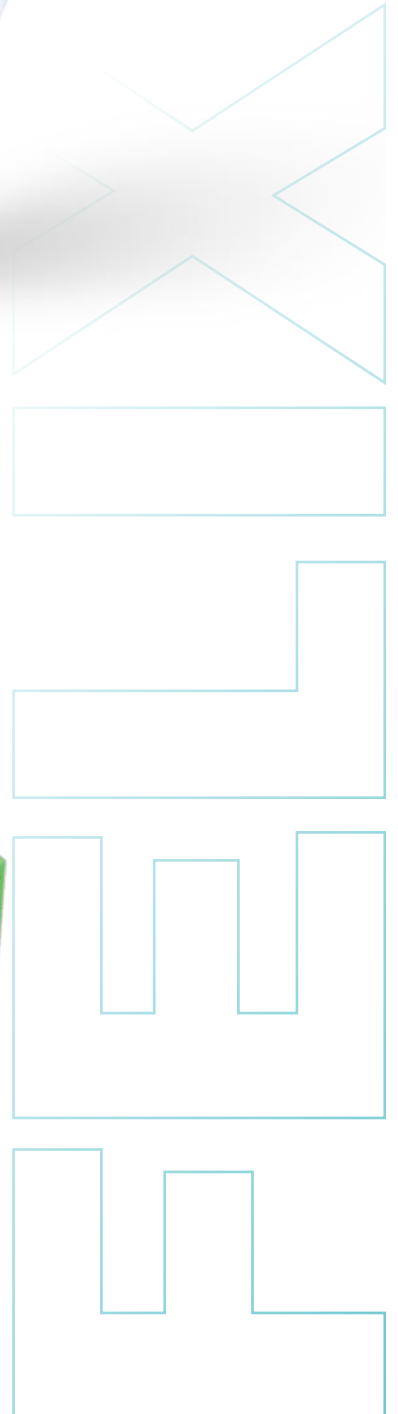
+RMR /M/100531	4g syringe A1
+RMR /M/100532	4g syringe A2
+RMR /M/100533	4g syringe A3
+RMR /M/100566	4g syringe B1
+RMR /M/100567	4g syringe B2
+RMR /M/100568	4g syringe MILKY BLEACH

INDICATIONS

- Direct anterior and posterior restorations in Black's classes I, II, III, IV, and V cavities.
- Inlays, onlays and veneers
- Extended fissure sealing in molars and premolars
- Core build-up
- Splinting loose teeth
- Adjusting the contours and shades to improve aesthetics

INFORMATION ON PROCESSING

Light intensity for curing	≥ 1200 mW/cm ²
Wavelength for curing	350-500 nm
Curing time for shades A1-A3.5, B1, B2 and Milky/Bleach	20 sec.
Curing time for shade A4	40 sec
Maximum layer thickness	2 mm

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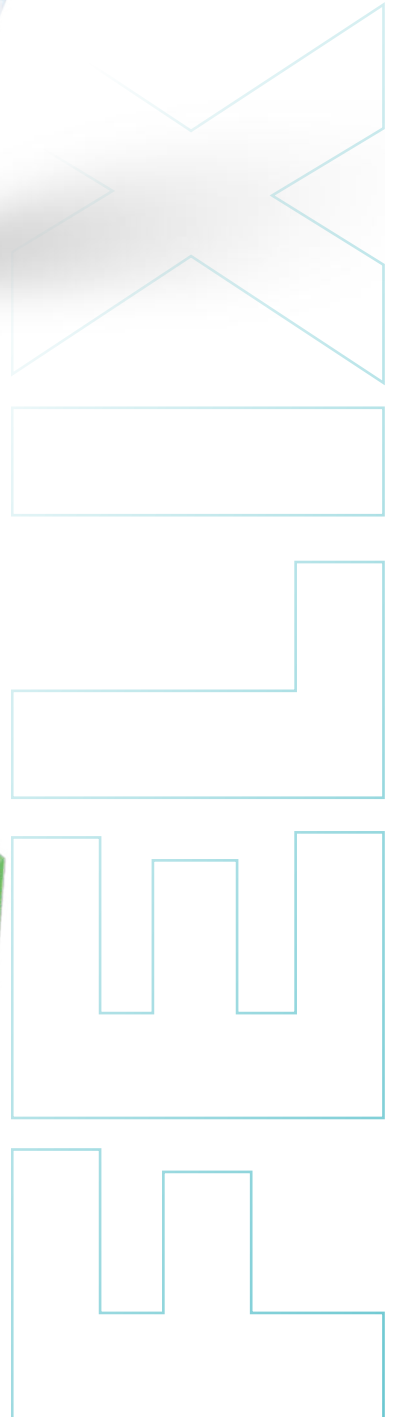
LUCAS Total Etching Gel

Etching gel suitable for etching dentine as well as enamel.

INTENDED PURPOSE

Dental etching gel is used to etch enamel and/or dentin.

DESCRIPTION

The LUCAS Total etching gel is a dental etching gel used to create micro-retentions on the tooth surface for the subsequent treatment.

Due to the coloring and stable gel consistency of the LUCAS Total Etching Gel, the application is a simple and easy to control.

PATIENT TARGET GROUP

LUCAS Total Etching Gel is used in all patients in whom micro retentions are to be created on their tooth surface for the further dental or orthodontic treatment.

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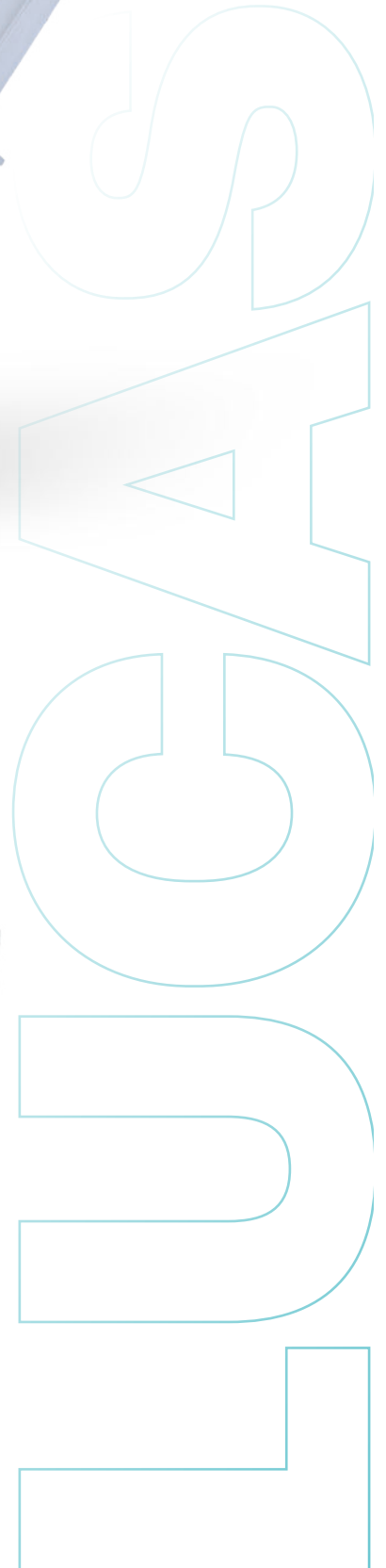
MARCUS Bleaching Set

DESCRIPTION

Marcus Bleaching Set is a bleaching set intended for medical in-office use only. The material is used by the dentist to brighten individual teeth. Marcus Bleaching Set is ideally suited to brightening devital teeth including endodontically treated teeth. The 4:1 auto mix syringe (cartridge) contains bleaching gel and activator. One dual syringe contains 5 g of material. The hydrogen peroxide content of the ready mixed gel is 35%.

PERFORMANCE CHARACTERISTICS

Determine the tooth color by comparing it with the Vita Shade Guide. To evaluate the colors, please resort the Vita Shade Guide as follows.

Values sorted by hue:

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Sorted according to brightness value:

B1 A1 B2 D2 A2 C1 C2 D3 A3 D4 B3 A3,5 B4 C3 A4 C4 B2-

B1 = Values are increased twofold

A3- A2 = Values are increased fourfold

PACKAGING

+RMR /M/100547	5g Bleaching gel
+RMR /M/100548	2ml Post care gel
+RMR /M/100549	5g Gingiva Barrier
	3 application tips
	3 mixing tips

INDICATIONS

- Brightening of discolored ,devital teeth
- Brightening of discolored teeth caused by disease, by injury or iatrogenically, as an alternative to a crown or veneer
- Treatment of tooth discolorations caused by prescription drugs (e.g.tetracycline)

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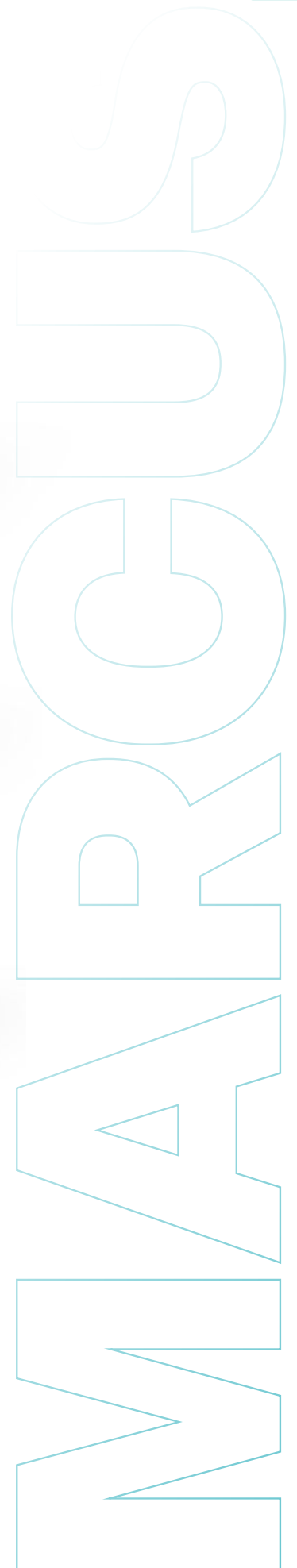
KLAUS ROOT CEM VISCOUS

Dual-curing, highly radio-opaque flowable luting cement on composite basis.

DESCRIPTION

Klaus Root Cem Viscous is a dual-curing attachment composite for a natural tooth. The material is fluorescent and radiopaque and is used to permanently attach indirect restorations to the tooth. This prevents the restoration from becoming dislodged whilst at the same time filling the minimum gap between dental preparation (or implant abutment) and restoration.

COMPOSITION

Glass powder, diurethane dimethacrylate, prepolymer, tetramethylene dimethacrylate, silicon dioxide, dibenzoyl peroxide
Filler content: 75% by weight, inorganic fillers (0.005 - 5.0 µm)

MIXING RATIO

With the automix cartridge and the mixing tip, KLAUS Root Cem Viscous is always available in the optimal mixing ratio of 1:1

PACKAGING

+RMR	/M/100550	2x8g + 10 cannules
------	-----------	--------------------

INDICATIONS

- The adhesives fixation of indirect restorations such as veneers, inlays, onlays, crown and bridges. The material is suited to fixing restorations made from glass ceramics, zirconium dioxide, composite, zirconia-reinforced composite, high-precious metal, non-precious metal, titanium.
- The adhesive fixation of root pins
- Building up stumps (as core build up material)

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**REMAR TECHNIK GmbH**